



ISORAPID® OP-Forte AF

Disinfectant concentrate for the disinfection and cleaning of non-invasive medical devices with sensitive surfaces.



Intended Purpose



ISORAPID® OP-Forte AF is a gentle but powerful disinfectant concentrate for the disinfection and cleaning of non-invasive medical devices with sensitive surfaces such as radiology equipments. ISORAPID® OP-Forte AF has a fruity scent and is compatible with acrylic glass, polycarbonate (PC), and PVC. It is free of aldehydes, phenols, and caustic potash.

Intended User



The product is intended for use by dental or healthcare professionals with specific knowledge about the disinfection of medical devices. The product is not to be used directly on patients.

Instructions for Use



For your protection, wear a laboratory gown, household gloves and safety goggles. Prepare the working solution according to the dosing table. Completely wipe or immerse the surface of the medical device to be disinfected and leave to act. Adhere to contact time. If necessary, dry the surface with a clean single-use cloth. Before first time use, test material compatibility by applying the disinfectant to a small inconspicuous area. A list of compatible and incompatible materials is available on our website. If you are uncertain as to the material composition of the medical device, contact the manufacturer before use. All serious accidents occurring in relation to the Medical Device should be reported to the manufacturer or the competent authority.

Limitation



FOR PROFESSIONAL USE ONLY. Not for use on invasive medical devices. Do not mix with other products. Make sure that all visible soiling has been removed before disinfection.

Handling



WARNING H315 Causes skin irritation. H410 Very toxic to aquatic life with long lasting effects. P280 Wear protective gloves/protective clothing/eye protection/face protection. P302 + P352 IF ON SKIN: Wash with plenty of soap and water. P332 + P313 If skin irritation occurs: Get medical advice/attention. P362 Take off contaminated clothing. P391 Collect spillage. P501 Dispose of contents/container in accordance with local and national regulations. For more information, please refer to Safety Data Sheet available on our website.

Storage



Store in cool, dry and well-ventilated facility. Keep out from direct sunlight, heat, and open flame. Storage temperature: < 30°C.



Physical Properties



Physical state: Clear, slightly viscous liquid	pH-value: 6.0-8.0
Colour: Green	Shelf life: 36 months (unopened container)
Scent: Lemon	

Composition in 100 g



100 g contains 10 g quaternary ammonium compounds, auxiliaries and water.

Effective Spectrum



Bactericidal	EN 13727:2012+A1:2013* EN 16615:2015 [^]	1% in 1 min
Yeasticidal	EN 13624:2013* EN 16615:2015 [^]	1% in 1 min
Enveloped viruses	EN 14476:2013+A1:2015*	1% in 5 min
Limited virucidal	EN 14476:2013+A1:2015 [^]	2% in 5 min

[^] Tested under low organic load / * Tested under high organic load

Dosing Table



	 Concentrate	+	 Water	 Time
2%	20 ml	+	980 ml	5 min

Compatibility



Acetal polymer	Compatible
Acrylic glass	Compatible
Acrylonitrile butadiene styrene	Compatible
Aluminium	Compatible with restriction
Artificial leather	Compatible
Chlorinated polyvinyl chloride	Compatible
Polyamide	Compatible
Polybutylene terephthalate	Compatible
Polycarbonate	Compatible
Polyetherimide	Compatible
Polyethylene	Compatible
Polyethylene terephthalate	Compatible
Polyphenylene oxides	Compatible
Polyphenylsulfone	Compatible
Polypropylene	Compatible



Polystyrene	Compatible
Polysulfone	Compatible
Polyvinyl chloride	Compatible
Polyvinylidene fluoride	Compatible
Silicone	Compatible
Thermoplastic elastomer	Compatible
Thermoplastic polyurethane	Compatible
Ultra-high-molecular-weight polyethylene	Compatible

Delivery Forms



CH-010005	250 ml bottle
CH-010059	1 litre bottle
CH-010060	2 litre bottle
CH-010065	5 litre canister

Legal Manufacturer, EU Representative, Importer, Distributor



Oro Clean Chemie AG • Allmendstrasse 21 • 8320 Fehraltorf • Switzerland
✉ info@oroclean.com • ☎ +41(0)44 226 44 44 • 🌐 www.oroclean.com



Oro Clean Chemie s.r.o • Vinohradská 2828/151 • Žižkov • 130 00 Praha 3 • Czech Republic
✉ info@oroclean.cz • 🌐 www.oroclean.cz