EN (English)

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INTENDED USERS: The product is intended for use by dental and healthcare professionals who have knowledge of the disinfection of medical devices. The product must not be used directly on patients.

EFFECTIVE SPECTRUM: Bactericidal (EN 13727, * EN 16151*) 60 s; tuberculocidal (EN 13438*) 60 s; mycobactericidal (EN 13438*) 60 s; yeasticidal (EN 13624*, EN 16151*) 60 s; virucidal against enveloped viruses (EN 14476*) 15 s; virucidal (EN 14476*) 60 s

* Tested under low organic load / * Tested under high organic load

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INSTRUCTIONS FOR USE: For your protection, wear disposable gloves and safety goggles. Completely moisten and wipe the entire surface of the medical device. Leave the disinfectant adhered to the surface of the medical devices that come into contact with skin using a water-soaked single-use cloth to prevent the build-up of biocides. Use an alcohol-compatible materials only. Before first use, check material compatibility on a small, inconspicuous area. If not compatible and the disinfectant material is available on our website. If you are unsure about the material composition of the medical device to be disinfected, contact the manufacturer before use. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority. The user and/or patient is established.

LIMITATIONS: FOR PROFESSIONAL USE ONLY. Not intended for use on invasive medical devices. Do not mix with other products. Ensure that all visible soiling has been removed before disinfection. Failure to adhere to these limitations may increase the risk of infection.

HANDLING: DANGER H226 Flammable liquid and vapour. H318 Causes serious eye damage. H336 May cause drowsiness or dizziness. P210 Keep away from heat, hot surfaces, sparks, open flames and all other ignition sources. Do not smoke. P233 Keep container tightly closed. P261 Avoid breathing fume/mist/vapours/spray. P280 Wear protective gloves/protective clothing/eye protection/face protection. P305 + P351 + P338 If in EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P308 + P313 If exposed or contents of container in accordance with local and national regulations. Further information can be found in the Safety Data Sheet available on orcleoan.com.

STORAGE: Store in a cool, dry and well-ventilated place. Keep away from direct sunlight, heat, and open flame. Storage temperature: < 30 °C (86 °F). Shelf life: 24 months.

COMPOSITION: 100 g contains 20 g ethanol, 28 g 1-propanol, 0.056 g quaternary ammonium compounds, auxiliaries and water.

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