



Ready-to-use disinfectant for the quick disinfection of dental impressions.

actisolve® imprespray is a ready-to-use solution for the rapid disinfection of dental impressions and impression trays. actisolve® imprespray does not affect the dimensional stability of dental impressions made of alginate, polyether, polysulfide, A-silicone, and C-silicone.

INSTRUCTIONS FOR USE

For your protection, wear disposable gloves and safety goggles. Completely moisten the dental impression and impression tray and leave disinfectant to act. Adhere to, but do not exceed, the indicated contact time. After disinfection, rinse the dental impressions thoroughly under running water of at least drinking-water quality for 10 seconds. Use on alcohol-compatible materials only. A list of compatible and incompatible materials is available on our website. If you are unsure about the material composition of the medical device to be disinfected, contact the manufacturer before use. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

COMPATIBILITY

Acetal polymer	Compatible
Acrylic glass	Not compatible
Acrylonitrile butadiene styrene	Compatible
Alginate	Compatible
Aluminium	Compatible
Artificial leather	Compatible
A-silicone	Compatible
Chlorinated polyvinyl chloride	Compatible
C-silicone	Compatible
Latex	Compatible
Polyamide	Compatible
Polybutylene terephthalate	Compatible
Polycarbonate	Compatible
Polyether	Compatible
Polyetherimide	Compatible
Polyethylene	Compatible
Polyethylene terephthalate	Compatible
Polyphenylene oxides	Compatible
Polyphenylsulfone	Compatible
Polypropylene	Compatible
Polystyrene	Compatible
Polysulfide	Compatible
Polysulfone	Compatible
Polyvinyl chloride	Compatible
Polyvinylidene fluoride	Compatible
Rhodium	Compatible
Silicone	Compatible
Thermoplastic elastomer	Compatible
Thermoplastic polyurethane	Compatible
Ultra-high-molecular-weight polyethylene	Compatible

HANDLING

DANGER H226 Flammable liquid and vapour. H318 Causes serious eye damage. H336 May cause drowsiness or dizziness. P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P233 Keep container tightly closed. P261 Avoid breathing fume/mist/vapours/spray. P280 Wear protective gloves/protective clothing/eye protection/face protection. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P501 Dispose of contents/ container in accordance with local and national regulations. Further

information can be found in the Safety Data Sheet available on udm.swiss.

EFFECTIVE SPECTRUM / EXPOSURE TIMES

Bactericidal	60 sec
Antibiotic-resistant bacteria	60 sec
Tuberculocidal	60 sec
Mycobactericidal	60 sec
Yeasticidal	60 sec
Drug-resistant yeasts	60 sec
Virucidal against enveloped viruses	15 sec
Bovine viral diarrhoea virus (BVDV)	15 sec
Coronavirus	15 sec
Hepatitis B virus	15 sec
Hepatitis C virus	15 sec
Herpes simplex virus	15 sec
Human immunodeficiency virus (HIV)	15 sec
Influenza A virus	15 sec
Vaccinia virus	15 sec
Virucidal	60 sec
Adenovirus	60 sec
Norovirus	30 sec
Poliovirus	60 sec

Complies with EN 13624, EN 13727, EN 14348, EN 14476, EN 14561, EN 14562, EN 14563 and EN 17111.

COMPOSITION

100 g contains 20 g ethanol, 28 g 1-propanol, 0.056 g quaternary ammonium compounds, auxiliaries and water.

PHYSICAL PROPERTIES

Physical state:	Clear, non-viscous liquid
Colour:	Colourless
Scent:	Of alcohol, lemon
pH-value:	Neutral
Shelf life:	48 months

STORAGE

Store in a cool, dry and well-ventilated place. Keep away from direct sunlight, heat, and open flame. Storage temperature: < 30 °C. Dispose of the product after expiry date.

DELIVERY FORMS

AC-071046 | 1 litre bottle

LEGAL MANUFACTURER



United Disinfectant Manufacturers AG
Dr. Grass-Strasse 12
9490 Vaduz
Principality of Liechtenstein
Telephone: +423 237 15 03
info@udm.li
www.udm.li