




Non-foaming concentrate for the cleaning and deodorisation of dental suction units.

actisolve® aspiclean is a concentrate for the cleaning and deodorisation of dental suction units, spittoon bowls and amalgam separators. It is suitable for weekly cleaning.

INSTRUCTIONS FOR USE

For your protection, wear a laboratory gown, household gloves and safety goggles. Clean the suction unit weekly or in accordance with local regulations. Rinse the suction unit thoroughly with water after surgical procedures and prior to cleaning. Prepare 2 litres of working solution according to the dosing table. Aspirate 1.5 litres of the working solution and pour the remaining 500 ml down the spittoon bowl. Leave to act. Rinse thoroughly with water after cleaning. Not compatible with aluminium, copper, galvanized steel, and polyurethane. A list of compatible and incompatible materials is available on our website. If you are unsure about the material composition of the medical device to be disinfected, contact the manufacturer before use. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

DOSING

	Concentrate	+	Water
2%	40 ml		1960 ml

COMPATIBILITY

- Acrylic glass
- Acrylonitrile butadiene styrene
- Chlorinated polyvinyl chloride
- Liquid crystal polymer
- Polybutylene terephthalate
- Polycarbonate
- Polyether ether ketone
- Polyetherimide
- Polyethylene
- Polyethylene terephthalate
- Polyethylene terephthalate glycol
- Polyoxymethylene
- Polyphenylene oxide
- Polyphenylene sulfide
- Polyphenylsulfone
- Polypropylene
- Polystyrene
- Polysulfone
- Polyvinyl chloride
- Polyvinylidene fluoride
- Silicone
- Thermoplastic elastomer
- Thermoplastic polyurethane
- Ultra-high-molecular-weight polyethylene

[illegible]

HANDLING

DANGER H290 May be corrosive to metals. H315 Causes skin irritation. H318 Causes serious eye damage. P280 Wear protective gloves/protective clothing/eye protection/face protection. P302 + P352 IF ON SKIN: Wash with plenty of soap and water. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P310 Immediately call a POISON CENTER or doctor/physician. P332 + P313 If skin irritation occurs: Get medical advice/attention. P362 Take off contaminated clothing. Further information can be found in the Safety Data Sheet available on udm.swiss.

COMPOSITION

100 g contains 9.75 g alkaline cleaning compounds, anti-foaming agents, auxiliaries and water.

PHYSICAL PROPERTIES

Physical state:	Clear, non-viscous liquid
Colour:	Yellow
Scent:	Peppermint
pH-value:	12.5-14.5
Shelf life:	36 months

STORAGE

Store in a cool, dry and well-ventilated place. Keep away from direct sunlight, heat, and open flame. Storage temperature: < 30 °C. Dispose of the product after expiry date.

DELIVERY FORMS

AC-071005	250 ml bottle
AC-071120	1 litre bottle
AC-071125	2 litre bottle

LEGAL MANUFACTURER



United Disinfectant Manufacturers AG
Dr. Grass-Strasse 12
9490 Vaduz
Principality of Liechtenstein
Telephone: +423 237 15 03
info@udm.li
www.udm.li