



# DENTIRO® Omni

**Surface disinfectant with a low alcohol content for medical devices.**



## Intended Purpose



DENTIRO® Omni is a surface disinfectant with a low alcohol content for the rapid disinfection of surfaces of non-invasive medical devices such as operating tables, gurneys, IV poles, and dental chairs.

## Intended User



The product is intended for use by dental or healthcare professionals with knowledge about the disinfection of medical devices. The product is not to be used directly on patients.

## Instructions for Use



For your protection, wear disposable gloves and safety goggles. Completely moisten and wipe the entire surface of the medical device. Leave disinfectant to act and adhere to contact time. Wipe medical devices, which come into contact with skin with a water-soaked single-use cloth to prevent the build-up of biocides. Use on alcohol compatible materials only. Before first time use, test material compatibility by applying the disinfectant to a small inconspicuous area. A list of compatible and incompatible materials is available on our website. If you are uncertain as to the material composition of the medical device, contact the manufacturer before use. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## Limitation



FOR PROFESSIONAL USE ONLY. Not for use on invasive medical devices. Do not mix with other products. Make sure that all visible soiling has been removed before disinfection. Failure to adhere to these limitations may lead to a potential loss of product efficacy resulting in an increased risk of infection.

## Handling



DANGER H225 Highly flammable liquid and vapour. H319 Causes serious eye irritation. P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P233 Keep container tightly closed. P280 Wear protective gloves/protective clothing/eye protection/face protection. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P337 + P313 If eye irritation persists: Get medical advice/attention. P501 Dispose of contents/container in accordance with local and national regulations. Further information can be found in the Safety Data Sheet available on [oroclean.com](http://oroclean.com).



## Storage



Store in cool, dry and well-ventilated facility. Keep out from direct sunlight, heat, and open flame. Storage temperature: < 30°C. Dispose product after expiry date.

## Physical Properties



Physical state: Clear, non-viscous liquid	pH-value: Neutral
Colour: Colourless	Shelf life: 24 months (unopened container)
Scent: Of alcohol	

## Composition in 100 g



100 g contains 7.5 g ethanol, 15 g isopropanol, 0.76 g quaternary ammonium compounds, auxiliaries and water.

## Effective Spectrum



Bactericidal	EN 13727* EN 16615^	60 sec
Yeasticidal	EN 13624* EN 16615^	60 sec
Virucidal against enveloped viruses	EN 14476*	15 sec
Limited spectrum virucidal	EN 14476^	5 min

^ Tested under low organic load / \* Tested under high organic load

## Compatibility



Acetal polymer	Compatible
Acrylic glass	Compatible
Acrylonitrile butadiene styrene	Compatible
Artificial leather	Compatible
Chlorinated polyvinyl chloride	Compatible
Polyamide	Compatible
Polybutylene terephthalate	Compatible
Polycarbonate	Compatible
Polyetherimide	Compatible
Polyethylene	Compatible
Polyethylene terephthalate	Compatible
Polyphenylene oxides	Compatible
Polyphenylsulfone	Compatible
Polypropylene	Compatible
Polystyrene	Compatible
Polysulfone	Compatible
Polyvinyl chloride	Compatible



## Delivery Forms





CH-032013	250 ml bottle
CH-032016	1 litre bottle
CH-032020	2 litre bottle
CH-032025	5 litre canister




## Legal Manufacturer



United Disinfectant Manufacturers AG • Dr. Grass-Strasse 12 • 9490 Vaduz • Principality of Liechtenstein

 +423 237 15 03 •  [info@udm.li](mailto:info@udm.li) •  [www.udm.li](http://www.udm.li)

A subsidiary of Oro Clean Chemie AG • Allmendstrasse 21 • 8320 Fehraltorf • Switzerland

 +41 (0)44 226 44 44 •  [info@oroclean.com](mailto:info@oroclean.com) •  [www.oroclean.com](http://www.oroclean.com)

Made in Switzerland