



DENTIRO® Omni Wipes

Disinfectant wipes with a low alcohol content for medical devices.



Intended Purpose



DENTIRO® Omni Wipes are ready-to-use disinfectant wipes with a low alcohol content. The disinfectant wipes are suitable for the rapid disinfection of surfaces of non-invasive medical devices such as operating tables, gurneys, IV poles, and dental chairs.

Intended User



The product is intended for use by dental or healthcare professionals with knowledge about the disinfection of medical devices. The product is not to be used directly on patients.

Instructions for Use



Write the use-by date after opening on the dispenser tub. For your protection, wear disposable gloves and safety goggles. Insert the refill pouch into the designated dispenser tub. Before first use, cut open the refill pouch. Pull the first wipe from the centre of the roll through the star-shaped opening in the lid. Close the dispenser tub firmly after each use. Completely moisten and wipe the entire surface of the medical device. Leave the disinfectant to act and adhere to contact time. Wipe medical devices that come into contact with skin using a water-soaked single-use cloth to prevent the build-up of biocides. Use on alcohol-compatible materials only. Before first use, check material compatibility on a small, inconspicuous area. A list of compatible and incompatible materials is available on our website. If you are unsure about the material composition of the medical device to be disinfected, contact the manufacturer before use. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Limitation



FOR PROFESSIONAL USE ONLY. Not intended for use on invasive medical devices. Do not mix with other products. Ensure that all visible soiling has been removed before disinfection. Do not re-use the wipes. Failure to adhere to these limitations may reduce the efficacy of the product, thereby increasing the risk of infection.

Handling



DANGER H228 Flammable solid. H319 Causes serious eye irritation. P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P233 Keep container tightly closed. P280 Wear protective gloves/protective clothing/eye protection/face protection. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P337 + P313 If eye irritation persists: Get medical advice/attention. P501 Dispose of contents/container in accordance with local and national regulations. Further information can be found in the Safety Data Sheet available on oroclean.com.



Storage



Store in a cool, dry and well-ventilated place. Keep away from direct sunlight, heat, and open flame. Storage temperature: < 30 °C. Dispose of the refill pouch 7 days after opening. Write the use-by date after opening on the dispenser tub.

Physical Properties



Physical state: Liquid absorbed on fleece	pH-value: Neutral
Colour: Colourless	Shelf life: 24 months (unopened container)
Scent: Of alcohol	

Composition in 100 g



100 g moisturizing solution contains 7.5 g ethanol, 15 g isopropanol, 0.76 g quaternary ammonium compounds, auxiliaries and water.

Effective Spectrum



Bactericidal	EN 13727* EN 16615^	60 sec
Yeasticidal	EN 13624* EN 16615^	60 sec
Virucidal against enveloped viruses	EN 14476*	15 sec
Limited spectrum virucidal	EN 14476^	5 min

^ Tested under low organic load / * Tested under high organic load

Compatibility



Acetal polymer	Compatible
Acrylic glass	Compatible
Acrylonitrile butadiene styrene	Compatible
Artificial leather	Compatible
Chlorinated polyvinyl chloride	Compatible
Polyamide	Compatible
Polybutylene terephthalate	Compatible
Polycarbonate	Compatible
Polyetherimide	Compatible
Polyethylene	Compatible
Polyethylene terephthalate	Compatible
Polyphenylene oxides	Compatible
Polyphenylsulfone	Compatible
Polypropylene	Compatible
Polystyrene	Compatible



Polysulfone	Compatible
Polyvinyl chloride	Compatible
Polyvinylidene fluoride	Compatible
Silicone	Compatible
Thermoplastic elastomer	Compatible
Thermoplastic polyurethane	Compatible
Ultra-high-molecular-weight polyethylene	Compatible

Delivery Forms






CH-043014 Ultra Soft, Refill pouch (19.3 x 21 cm), 120 pcs, classic scent




Legal Manufacturer



United Disinfectant Manufacturers AG • Dr. Grass-Strasse 12 • 9490 Vaduz • Principality of Liechtenstein

 +423 237 15 03 •  info@udm.li •  www.udm.li

A subsidiary of Oro Clean Chemie AG • Allmendstrasse 21 • 8320 Fehraltorf • Switzerland

 +41 (0)44 226 44 44 •  info@oroclean.com •  www.oroclean.com

Made in Switzerland