



DENTIRO® Opti Wipes

Disinfectant wipes with a high alcohol content for residue-free disinfection of medical devices.



Intended Purpose



DENTIRO® Opti Wipes are ready-to-use disinfectant wipes with a high alcohol content for the rapid and residue-free disinfection of alcohol-resistant surfaces of non-invasive medical devices such as operating tables and IV poles.

Intended User



The product is intended for use by dental and healthcare professionals who have knowledge of the disinfection of medical devices. The product must not be used directly on patients.

Instructions for Use



Write the use-by date after opening on the dispenser tub. For your protection, wear disposable gloves and safety goggles. Insert the refill pouch into the designated dispenser tub. Before first use, cut open the refill pouch. Pull the first wipe from the centre of the roll through the star-shaped opening in the lid. Close the dispenser tub firmly after each use. Completely moisten and wipe the entire surface of the medical device. Leave the disinfectant to act and adhere to contact time. Use on alcohol-compatible materials only. Before first use, check material compatibility on a small, inconspicuous area. A list of compatible and incompatible materials is available on our website. If you are unsure about the material composition of the medical device to be disinfected, contact the manufacturer before use. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Limitation



FOR PROFESSIONAL USE ONLY. Not intended for use on invasive medical devices. Do not mix with other products. Ensure that all visible soiling has been removed before disinfection. Do not re-use the wipes. Failure to adhere to these limitations may reduce the efficacy of the product, thereby increasing the risk of infection.

Handling



DANGER H226 Flammable liquid and vapour. H318 Causes serious eye damage. H336 May cause drowsiness or dizziness. P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P233 Keep container tightly closed. P261 Avoid breathing fume/mist/vapours/spray. P280 Wear protective gloves/protective clothing/eye protection/face protection. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P501 Dispose of contents/container in accordance with local and national regulations. Further information can be found in the Safety Data Sheet available on oroclean.com.



Storage



Store in a cool, dry and well-ventilated place. Keep away from direct sunlight, heat, and open flame. Storage temperature: < 30 °C. Dispose of the refill pouch 30 days after opening. Write the use-by date after opening on the dispenser tub.

Physical Properties



Physical state: Liquid absorbed on fleece	pH-value: Neutral
Colour: Colourless	Shelf life: 24 months (unopened container)
Scent: Of alcohol	

Composition in 100 g



100 g moisturizing solution contains 25 g ethanol, 35 g 1-propanol, auxiliaries and water.

Effective Spectrum



Bactericidal	EN 13727* EN 16615#	60 sec
Tuberculocidal	EN 14348^	60 sec
Mycobactericidal	EN 14348^	60 sec
Yeasticidal	EN 13624* EN 16615^	60 sec
Virucidal against enveloped viruses	EN 14476*	15 sec
Limited spectrum virucidal	EN 14476^	30 sec

^ Tested under low organic load / * Tested under high organic load / # Tested under low and high organic load

Compatibility



Acetal polymer	Compatible
Acrylic glass	Not compatible
Acrylonitrile butadiene styrene	Compatible
Artificial leather	Compatible
Chlorinated polyvinyl chloride	Compatible
Polyamide	Compatible
Polybutylene terephthalate	Compatible
Polycarbonate	Compatible
Polyetherimide	Compatible
Polyethylene	Compatible
Polyethylene terephthalate	Compatible
Polyphenylene oxides	Compatible
Polyphenylsulfone	Compatible
Polypropylene	Compatible
Polystyrene	Compatible
Polysulfone	Compatible



Polyvinyl chloride	Compatible
Polyvinylidene fluoride	Compatible
Silicone	Compatible
Thermoplastic elastomer	Compatible
Thermoplastic polyurethane	Compatible
Ultra-high-molecular-weight polyethylene	Compatible

Delivery Forms



CH-043022	Refill pouch (14.5 x 20 cm), 120 pcs, classic scent
CH-043023	Refill pouch (14.5 x 20 cm), 120 pcs, lemon scent
CH-043025	Jumbo, Refill pouch (14.5 x 40 cm), 100 pcs, classic scent
CH-043026	Jumbo, Refill pouch (14.5 x 40 cm), 100 pcs, lemon scent
CH-043021	Smart Saver, Refill pouch (14.5 x 20 cm), 250 pcs, lemon scent
CH-043040	Smart Saver, Refill pouch (14.5 x 20 cm), 120 pcs, lemon scent
CH-043024	Ultra Soft, Refill pouch (19.3 x 21 cm), 120 pcs, lemon scent

Legal Manufacturer



United Disinfectant Manufacturers AG • Dr. Grass-Strasse 12 • 9490 Vaduz • Principality of Liechtenstein

+423 237 15 03 • info@udm.li • www.udm.li

A subsidiary of Oro Clean Chemie AG • Allmendstrasse 21 • 8320 Fehraltorf • Switzerland

+41 (0)44 226 44 44 • info@oroclean.com • www.oroclean.com

Made in Switzerland