



ORO CLEAN® Plus

Concentrate for the disinfection, cleaning and deodorisation of dental suction units.



Intended Purpose



ORO CLEAN® Plus is a concentrate for the disinfection and cleaning of dental suction units and spittoon bowls.

Intended User



The product is intended for use by dental and healthcare professionals who have knowledge of the disinfection of medical devices. The product must not be used directly on patients.

Instructions for Use



For your protection, wear a laboratory gown, household gloves and safety goggles. Disinfect the suction unit daily or in accordance with local regulations. Rinse the suction unit thoroughly with water after surgical procedures and prior to disinfection. Prepare 2 litres of working solution according to the dosing table. Aspirate 1.5 litres of the working solution and pour the remaining 500 ml down the spittoon bowl. Leave to act. Rinse thoroughly with water after disinfection. Not compatible with aluminium, copper, galvanized steel, and polyurethane. A list of compatible and incompatible materials is available on our website. If you are unsure about the material composition of the medical device to be disinfected, contact the manufacturer before use. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Limitation



FOR PROFESSIONAL USE ONLY. Not intended for use on invasive medical devices. Do not mix with other products. Ensure that all visible soiling has been removed before disinfection. Failure to adhere to these limitations may reduce the efficacy of the product, thereby increasing the risk of infection.

Handling



DANGER H290 May be corrosive to metals. H314 Causes severe skin burns and eye damage. H410 Very toxic to aquatic life with long lasting effects. P280 Wear protective gloves/protective clothing/eye protection/face protection. P301 + P330 + P331 IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. P303 + P361 + P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P391 Collect spillage. P501 Dispose of contents/container in accordance with local and national regulations. Further information can be found in the Safety Data Sheet available on oroclean.com.

Storage



Store in a cool, dry and well-ventilated place. Keep away from direct sunlight, heat, and open flame. Storage temperature: < 30 °C. Dispose of the product after expiry date.



Physical Properties



Physical state: Clear, slightly-viscous liquid	pH-value: 10.0 - 12.0
Colour: Yellow	Shelf life: 48 months
Scent: Aromatic	

Composition in 100 g



100 g contains 2.475 g N-(3-aminopropyl)-N-dodecylpropane-1,3-diamine, 7.5 g quaternary ammonium compounds, anti-foaming agents, auxiliaries and water.

Effective Spectrum



Bactericidal	EN 13727*, EN 14561*	2% in 5 min
Yeasticidal	EN 13624*, EN 14562*	2% in 5 min
Virucidal against enveloped viruses	EN 17111 [^]	2% in 5 min
Virucidal	EN 14476 [^] , EN 17111 [^]	2% in 60 min

[^] Tested under low organic load / * Tested under high organic load

Dosing Table



	 Concentrate	+	 Water
2%	40 ml	+	1960 ml

Compatibility



Acrylonitrile butadiene styrene	Compatible
Latex	Compatible
Polyethylene	Compatible
Polypropylene	Compatible
Polystyrene	Compatible
Polyvinyl chloride	Compatible

Delivery Forms



CH-071105	250 ml bottle
CH-071020	2 litre bottle
CH-071025	5 litre canister

Legal Manufacturer



United Disinfectant Manufacturers AG • Dr. Grass-Strasse 12 • 9490 Vaduz • Principality of Liechtenstein



+423 237 15 03 •



info@udm.li •



www.udm.li

A subsidiary of Oro Clean Chemie AG • Allmendstrasse 21 • 8320 Fehraltorf • Switzerland



+41 (0)44 226 44 44 •



info@oroclean.com •



www.oroclean.com

Made in Switzerland