



ASEPTOPRINT® Liquid

Concentrate for the disinfection of dental impressions.



Intended Purpose



ASEPTOPRINT® Liquid is a concentrate for the disinfection of dental impressions made of alginate, polyether, polysulfide, A-silicone, and C-silicone. ASEPTOPRINT® Liquid does not affect the dimensional stability of dental impressions.

Intended User



The product is intended for use by dental and healthcare professionals who have knowledge of the disinfection of medical devices. The product must not be used directly on patients.

Instructions for Use



For your protection, wear disposable gloves and safety goggles. Prepare the working solution according to the dosing table. Immerse the dental impression and impression tray completely in the disinfectant solution. Adhere to, but do not exceed, the indicated contact time. After disinfection, rinse the dental impressions thoroughly under running water of at least drinking-water quality for 10 seconds. Replace the working solution daily or when visibly contaminated. A list of compatible and incompatible materials is available on our website. If you are unsure about the material composition of the medical device to be disinfected, contact the manufacturer before use. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Limitation



FOR PROFESSIONAL USE ONLY. Do not mix with other products. Ensure that all visible soiling has been removed before disinfection. Failure to adhere to these limitations may reduce the efficacy of the product, thereby increasing the risk of infection.

Handling



DANGER H302 Harmful if swallowed. H314 Causes severe skin burns and eye damage. H410 Very toxic to aquatic life with long lasting effects. P280 Wear protective gloves/protective clothing/eye protection/face protection. P301 + P312 IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. P301 + P330 + P331 IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. P303 + P361 + P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P501 Dispose of contents/container in accordance with local and national regulations. Further information can be found in the Safety Data Sheet available on oroclean.com.

Storage



Store in a cool, dry and well-ventilated place. Keep away from direct sunlight, heat, and open flame. Storage temperature: < 30 °C. Dispose of the product after expiry date.



Physical Properties



Physical state: Clear, slightly viscous liquid	pH-value: 10.5-12.5
Colour: Light yellow	Shelf life: 36 months (unopened container)
Scent: Aromatic	

Composition in 100 g



100 g contains 30 g benzalkonium chloride, auxiliaries and water.

Effective Spectrum






Bactericidal	EN 13727*, EN 14561#	3% in 3 min
Yeasticidal	EN 13624*, EN 14562#	3% in 3 min
Virucidal against enveloped viruses	EN 17111 ^	3% in 3 min

^ Tested under low organic load / * Tested under high organic load / # Tested under low and high organic load

Dosing Table



	 Concentrate	+	 Water	 Time
3%	30 ml	+	970 ml	3 min

Compatibility



Acrylic glass	Compatible
Acrylonitrile butadiene styrene	Compatible
Alginate	Compatible
A-silicone	Compatible
C-silicone	Compatible
Polycarbonate	Compatible
Polyether	Compatible
Polyethylene	Compatible
Polyethylene terephthalate	Compatible
Polypropylene	Compatible
Polystyrene	Compatible
Polysulfide	Compatible
Polyvinyl chloride	Compatible
Stainless steel	Compatible
Thermoplastic elastomer	Compatible
Thermoplastic polyurethane	Compatible



Delivery Forms



CH-071041

1 litre bottle

Legal Manufacturer



United Disinfectant Manufacturers AG • Dr. Grass-Strasse 12 • 9490 Vaduz • Principality of Liechtenstein



+423 237 15 03 •



info@udm.li •



www.udm.li

A subsidiary of Oro Clean Chemie AG • Allmendstrasse 21 • 8320 Fehraltorf • Switzerland



+41 (0)44 226 44 44 •



info@oroclean.com •



www.oroclean.com

Made in Switzerland