



# ASEPTOPRINT® Spray

Ready-to-use disinfectant for the quick disinfection of dental impressions.



## Intended Purpose



ASEPTOPRINT® Spray is a ready-to-use product for the rapid disinfection of dental impressions and impression trays. ASEPTOPRINT® Spray does not affect the dimensional stability of dental impressions made of alginate, polyether, polysulfide, A-silicone, and C-silicone.

## Intended User



The product is intended for use by dental and healthcare professionals who have knowledge of the disinfection of medical devices. The product must not be used directly on patients.

## Instructions for Use



For your protection, wear disposable gloves and safety goggles. Completely moisten the dental impression and impression tray and leave disinfectant to act. Adhere to, but do not exceed, the indicated contact time. After disinfection, rinse the dental impressions thoroughly under running water of at least drinking-water quality for 10 seconds. Use on alcohol-compatible materials only. A list of compatible and incompatible materials is available on our website. If you are unsure about the material composition of the medical device to be disinfected, contact the manufacturer before use. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## Limitation



FOR PROFESSIONAL USE ONLY. Do not mix with other products. Ensure that all visible soiling has been removed before disinfection. Failure to adhere to these limitations may reduce the efficacy of the product, thereby increasing the risk of infection.

## Handling



DANGER H226 Flammable liquid and vapour. H318 Causes serious eye damage. H336 May cause drowsiness or dizziness. P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P233 Keep container tightly closed. P261 Avoid breathing fume/mist/vapours/spray. P280 Wear protective gloves/protective clothing/eye protection/face protection. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P501 Dispose of contents/container in accordance with local and national regulations. Further information can be found in the Safety Data Sheet available on oroclean.com.



## Storage



Store in a cool, dry and well-ventilated place. Keep away from direct sunlight, heat, and open flame. Storage temperature: < 30 °C. Dispose of the product after expiry date.

## Physical Properties



Physical state: Clear, non-viscous liquid	pH-value: Neutral
Colour: Colourless	Shelf life: 48 months
Scent: Of alcohol, lemon	

## Composition in 100 g



100 g contains 20 g ethanol, 28 g 1-propanol, 0.056 g quaternary ammonium compounds, auxiliaries and water.

## Effective Spectrum



Bactericidal	EN 13727* EN 14561*	60 sec
Tuberculocidal	EN 14348* EN 14563^	60 sec
Mycobactericidal	EN 14348* EN 14563^	60 sec
Yeasticidal	EN 13624* EN 14562^	60 sec
Virucidal against enveloped viruses	EN 17111^	15 sec
Virucidal	EN 14476^ EN 17111^	60 sec

^ Tested under low organic load / \* Tested under high organic load

## Compatibility



Acetal polymer	Compatible
Acrylic glass	Not compatible
Acrylonitrile butadiene styrene	Compatible
Alginate	Compatible
Aluminium	Compatible
Artificial leather	Compatible
A-silicone	Compatible
Chlorinated polyvinyl chloride	Compatible
C-silicone	Compatible
Polyamide	Compatible
Polybutylene terephthalate	Compatible
Polycarbonate	Compatible
Polyether	Compatible



Polyetherimide	Compatible
Polyethylene	Compatible
Polyethylene terephthalate	Compatible
Polyphenylene oxides	Compatible
Polyphenylsulfone	Compatible
Polypropylene	Compatible
Polystyrene	Compatible
Polysulfide	Compatible
Polysulfone	Compatible
Polyvinyl chloride	Compatible
Polyvinylidene fluoride	Compatible
Rhodium	Compatible
Silicone	Compatible
Thermoplastic elastomer	Compatible
Thermoplastic polyurethane	Compatible
Ultra-high-molecular-weight polyethylene	Compatible

## Delivery Forms



CH-071045	250 ml bottle
CH-071046	1 litre bottle

## Legal Manufacturer



United Disinfectant Manufacturers AG • Dr. Grass-Strasse 12 • 9490 Vaduz • Principality of Liechtenstein

+423 237 15 03 • [info@udm.li](mailto:info@udm.li) • [www.udm.li](http://www.udm.li)

A subsidiary of Oro Clean Chemie AG • Allmendstrasse 21 • 8320 Fehraltorf • Switzerland

+41 (0)44 226 44 44 • [info@oroclean.com](mailto:info@oroclean.com) • [www.oroclean.com](http://www.oroclean.com)

Made in Switzerland