



Disinfectant wipes with a low alcohol content for medical devices.

actisolve® alcolite wipes are ready-to-use disinfectant wipes with a low alcohol content for the disinfection of surfaces of non-invasive medical devices. The product can be safely used on sensitive medical devices such as treatment chairs with high-gloss surfaces. The disinfectant wipes are compatible with the most commonly used materials in medical devices as well as acrylic glass and artificial leather.

INSTRUCTIONS FOR USE

Write the use-by date after opening on the dispenser tub. For your protection, wear disposable gloves and safety goggles. Insert the refill pouch into the designated dispenser tub. Before first use, cut open the refill pouch. Pull the first wipe from the centre of the roll through the star-shaped opening in the lid. Close the dispenser tub firmly after each use. Completely moisten and wipe the entire surface of the medical device. Leave the disinfectant to act and adhere to contact time. Wipe medical devices that come into contact with skin using a water-soaked single-use cloth to prevent the build-up of biocides. Use on alcohol-compatible materials only. Before first use, check material compatibility on a small, inconspicuous area. A list of compatible and incompatible materials is available on our website. If you are unsure about the material composition of the medical device to be disinfected, contact the manufacturer before use. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

COMPATIBILITY

Acetal polymer	Compatible
Acrylic glass	Compatible
Acrylonitrile butadiene styrene	Compatible
Artificial leather	Compatible
Chlorinated polyvinyl chloride	Compatible
Latex	Compatible
Polyamide	Compatible
Polybutylene terephthalate	Compatible
Polycarbonate	Compatible
Polyetherimide	Compatible
Polyethylene	Compatible
Polyethylene terephthalate	Compatible
Polyphenylene oxides	Compatible
Polyphenylsulfone	Compatible
Polypropylene	Compatible
Polystyrene	Compatible
Polysulfone	Compatible
Polyvinyl chloride	Compatible
Polyvinylidene fluoride	Compatible
Silicone	Compatible
Thermoplastic elastomer	Compatible
Thermoplastic polyurethane	Compatible
Ultra-high-molecular-weight polyethylene	Compatible

HANDLING

WARNING H226 Flammable liquid and vapour. H319 Causes serious eye irritation. P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P233 Keep container tightly closed. P280 Wear protective gloves/protective clothing/eye protection/face protection. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P337 + P313 If eye irritation persists: Get medical advice/attention. P501 Dispose of contents/container in accordance with local and national regulations. Further information can be found in the Safety Data Sheet available on udm.swiss.

EFFECTIVE SPECTRUM / EXPOSURE TIMES

Bactericidal	60 sec
Antibiotic-resistant bacteria	60 sec
Yeasticidal	60 sec
Drug-resistant yeasts	60 sec
Virucidal against enveloped viruses	15 sec
Bovine viral diarrhea virus (BVDV)	15 sec
Coronavirus	15 sec
Hepatitis B virus	15 sec
Hepatitis C virus	15 sec
Herpes simplex virus	15 sec
Human immunodeficiency virus (HIV)	15 sec
Influenza A virus	15 sec
Vaccinia virus	15 sec
Limited spectrum virucidal	5 min
Adenovirus	5 min
Norovirus	5 min

Complies with EN 13624, EN 13727, EN 14476 and EN 16615.

COMPOSITION

100 g impregnating solution contains 7.5 g ethanol, 15 g isopropanol, 0.76 g quaternary ammonium compounds, auxiliaries and water.

PHYSICAL PROPERTIES

Physical state:	Liquid absorbed on fleece
Colour:	Colourless
Scent:	Of alcohol
pH-value:	Neutral
Shelf life:	24 months

DELIVERY FORMS

AC-043014	Soft, Refill pouch with 120 disinfectant wipes (19.3 x 21 cm), classic scent.
-----------	---

STORAGE

Store in a cool, dry and well-ventilated place. Keep away from direct sunlight, heat, and open flame. Storage temperature: < 30 °C. Dispose of the refill pouch 7 days after opening.

LEGAL MANUFACTURER



United Disinfectant Manufacturers AG
Dr. Grass-Strasse 12
9490 Vaduz
Principality of Liechtenstein
Telephone: +423 237 15 03
info@udm.li
www.udm.li