



# ORO CLEAN® Liquid

**Non-foaming concentrate for the cleaning and deodorisation of dental suction units.**



## Intended Purpose



ORO CLEAN® Liquid is a concentrate for the cleaning and deodorisation of dental suction units, spittoon bowls and amalgam separators. It is suitable for weekly cleaning.

## Intended User



The product is intended for use by dental and healthcare professionals who have knowledge of the disinfection of medical devices. The product must not be used directly on patients.

## Instructions for Use



For your protection, wear a laboratory gown, household gloves and safety goggles. Clean the suction unit weekly or as per local regulations. Rinse the suction unit thoroughly with water after surgical procedures and prior to cleaning. Prepare 2 litres of working solution according to the dosing table. Aspirate 1.5 litres of the working solution and pour the remaining 500 ml down the spittoon bowl. Leave to act. Rinse thoroughly with water after cleaning. Not compatible with aluminium, copper, galvanized steel and polyurethane. A list of compatible and incompatible materials is available on our website. If you are uncertain as to the material composition of the medical device, contact the manufacturer before use. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## Limitation



FOR PROFESSIONAL USE ONLY. Not intended for use on invasive medical devices. Do not mix with other products.

## Handling



DANGER H290 May be corrosive to metals. H315 Causes skin irritation. H318 Causes serious eye damage. P280 Wear protective gloves/protective clothing/eye protection/face protection. P302 + P352 IF ON SKIN: Wash with plenty of soap and water. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P310 Immediately call a POISON CENTER or doctor/physician. P332 + P313 If skin irritation occurs: Get medical advice/attention. P362 Take off contaminated clothing. Further information can be found in the Safety Data Sheet available on oroclean.com.

## Storage



Store in a cool, dry and well-ventilated place. Keep away from direct sunlight, heat, and open flame. Storage temperature: < 30 °C. Dispose of the product after expiry date.



## Physical Properties



Physical state: Clear, non-viscous liquid	pH-value: 12.5-14.5
Colour: Yellow	Shelf life: 60 months
Scent: Peppermint	

## Composition in 100 g



100 g contains 9.75 g alkaline cleaning compounds, anti-foaming agents, auxiliaries and water.

## Dosing Table



	 Concentrate	+	 Water
2%	40 ml	+	1960 ml

## Compatibility



Acrylic glass	Compatible
Acrylonitrile butadiene styrene	Compatible
Chlorinated polyvinyl chloride	Compatible
Liquid crystal polymer	Compatible
Polybutylene terephthalate	Compatible
Polycarbonate	Compatible
Polyether ether ketone	Compatible
Polyetherimide	Compatible
Polyethylene	Compatible
Polyethylene terephthalate	Compatible
Polyethylene terephthalate glycol	Compatible
Polyoxymethylene	Compatible
Polyphenylene oxides	Compatible
Polyphenylene sulfide	Compatible
Polyphenylsulfone	Compatible
Polypropylene	Compatible
Polystyrene	Not Compatible
Polysulfone	Compatible
Polyvinyl chloride	Compatible
Polyvinylidene fluoride	Compatible
Silicone	Compatible
Thermoplastic elastomer	Compatible
Thermoplastic polyurethane	Compatible
Ultra-high-molecular-weight polyethylene	Compatible

## Delivery Forms



CH-071005	250 ml bottle
CH-071120	2 litre bottle
CH-071125	5 litre canister



## Legal Manufacturer



United Disinfectant Manufacturers AG • Dr. Grass-Strasse 12 • 9490 Vaduz • Principality of Liechtenstein

 +423 237 15 03 •  [info@udm.li](mailto:info@udm.li) •  [www.udm.li](http://www.udm.li)

A subsidiary of Oro Clean Chemie AG • Allmendstrasse 21 • 8320 Fehraltorf • Switzerland

 +41 (0)44 226 44 44 •  [info@oroclean.com](mailto:info@oroclean.com) •  [www.oroclean.com](http://www.oroclean.com)

Made in Switzerland