



## Surface disinfectant with a high alcohol content for medical devices.

actisolve® alcoplus is a ready-to-use surface disinfectant with a high alcohol content supplemented by quaternary ammonium compounds for the rapid disinfection of alcohol-resistant surfaces of non-invasive medical devices such as operating tables.

### INSTRUCTIONS FOR USE

For your protection, wear disposable gloves and safety goggles. Completely moisten and wipe the entire surface of the medical device. Leave the disinfectant to act and adhere to contact time. Wipe medical devices that come into contact with skin using a water-soaked single-use cloth to prevent the build-up of biocides. Use on alcohol-compatible materials only. Before first use, check material compatibility on a small, inconspicuous area. A list of compatible and incompatible materials is available on our website. If you are unsure about the material composition of the medical device to be disinfected, contact the manufacturer before use. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### COMPATIBILITY

Acetal polymer	Compatible
Acrylic glass	Not compatible
Acrylonitrile butadiene styrene	Compatible
Artificial leather	Compatible
Chlorinated polyvinyl chloride	Compatible
Latex	Compatible
Polyamide	Compatible
Polybutylene terephthalate	Compatible
Polycarbonate	Compatible
Polyetherimide	Compatible
Polyethylene	Compatible
Polyethylene terephthalate	Compatible
Polyphenylene oxides	Compatible
Polyphenylsulfone	Compatible
Polypropylene	Compatible
Polystyrene	Compatible
Polysulfone	Compatible
Polyvinyl chloride	Compatible
Polyvinylidene fluoride	Compatible
Silicone	Compatible
Thermoplastic elastomer	Compatible
Thermoplastic polyurethane	Compatible
Ultra-high-molecular-weight polyethylene	Compatible

### HANDLING

**DANGER H225** Highly flammable liquid and vapour. **H318** Causes serious eye damage. **P210** Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **P233** Keep container tightly closed. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P303 + P361 + P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. **P305 + P351 + P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P501** Dispose of contents/container in accordance with local and national regulations. Further information can be found in the Safety Data Sheet available on [udm.swiss](http://udm.swiss).

### EFFECTIVE SPECTRUM / EXPOSURE TIMES

Bactericidal	60 sec
Antibiotic-resistant bacteria	60 sec
Yeasticidal	60 sec
Drug-resistant yeasts	60 sec
Fungicidal	5 min
Virucidal against enveloped viruses	15 sec
Bovine viral diarrhea virus (BVDV)	15 sec
Coronavirus	15 sec
Hepatitis B virus	15 sec
Hepatitis C virus	15 sec
Herpes simplex virus	15 sec
Human immunodeficiency virus (HIV)	15 sec
Influenza A virus	15 sec
Vaccinia virus	15 sec
Limited spectrum virucidal	30 sec
Adenovirus	30 sec
Norovirus	30 sec

Complies with EN 13624, EN 13727, EN 14476 and EN 16615.

### COMPOSITION

100 g contains 55 g ethanol, 10 g 1-propanol, 0.2 g quaternary ammonium compounds, auxiliaries and water.

### PHYSICAL PROPERTIES

Physical state:	Clear, non-viscous liquid
Colour:	Colourless
Scent:	Of alcohol
pH-value:	Neutral
Shelf life:	24 months

### STORAGE

Store in a cool, dry and well-ventilated place. Keep away from direct sunlight, heat, and open flame. Storage temperature: < 30 °C. Dispose of the product after expiry date.

### DELIVERY FORMS

AC-034005	250 ml bottle
AC-034013	1 litre bottle
AC-034020	2 litre bottle
AC-034025	5 litre canister

### LEGAL MANUFACTURER



United Disinfectant Manufacturers AG  
Dr. Grass-Strasse 12  
9490 Vaduz  
Principality of Liechtenstein  
Telephone: +423 237 15 03  
[info@udm.li](mailto:info@udm.li)  
[www.udm.li](http://www.udm.li)